

ONLY ONE

FLOWSAFE II® DISPOSABLE CPAP SYSTEM

Flow-Safe II® Disposable CPAP Therapy System

- Ideal for wherever CPAP therapy is appropriate
- Pre-hospital or in hospital use
- Compact and easy to store – no extra parts
- Color-coded manometer verifies delivered CPAP pressure
- Uses 50% Less Oxygen Consumption than the original Flow-Safe
- Comfortable head harness with quick disconnect clips for easy access to patient's face
- Forehead support reduces pressure on the nose and easily adjusts
- Easy Set up & Training – just attach to standard flowmeters/Flow determines CPAP pressure. Yellow Flow/CPAP chart on all devices illustrates flow rate with corresponding delivered CPAP pressures.



CONNECT TO FLOW SOURCE ONLY FLOWSAFE II

CPAP/PEEP (cm H ₂ O)	Flow (LPM)
5.0	8 - 9
7.5	10 - 12
10.0	13 - 14
13.0 (MAX)	FLUSH

www.mercurymed.com

ONLY ONE

FLOWSAFE II® DISPOSABLE CPAP SYSTEM

Flow-Safe II® Disposable CPAP Therapy System
Compared to Non-invasive Mechanical Ventilation



Contents lists available at ScienceDirect

American Journal of Emergency Medicine

journal homepage: www.elsevier.com/locate/ajem



Is the flow-safe disposable continuous positive airway pressure (CPAP) system as effective as non-invasive mechanical ventilation (NIMV) in the treatment of acute cardiogenic pulmonary oedema?

Ilhan UZ, MD^{a,*}, Güçlü Selahattin KIYAN, MD^a, Enver ÖZÇETE, MD^a, Sercan YALÇINLI, MD^a, Mehmet Birkan KORGAN, MD^a, Yusuf Ali ALTUNCI, MD^a, Murat ERSEL, MD^a, Funda Karbek AKARCA, MD^a, Oğuz YAVUZGİL, MD^b

^a Department of Emergency Medicine, Ege University Faculty of Medicine, Izmir, Turkey
^b Department of Cardiology, Ege University Faculty of Medicine, Izmir, Turkey



According to this recent American Journal of Emergency Medicine, For acute cardiogenic pulmonary oedema, (ACPO) patients, the Flow-Safe® Disposable CPAP device is “as effective as NIMV, in improving blood pressure, pulse, respiratory rate and blood gas.”

Flow-Safe II, Flow-Safe II EZ® and Flow-Safe II+® can be solutions for situations where non-invasive mechanical ventilation equipment is scarce or unavailable.



ORDERING INFORMATION

Part #	Description	Packaged
#10-57203	Flow-Safe II CPAP System w/o mask	5/Box
#10-57209	Flow-Safe II CPAP System w/ large adult deluxe mask w/ straight swivel port, head harness	5/Box
#10-57210	Flow-Safe II CPAP System w/ medium/small adult deluxe mask w/ straight swivel port, head harness	5/Box
#10-57211	Flow-Safe II CPAP System w/ child deluxe mask w/ straight swivel port, head harness	5/Box

* Other configurations available

Quest Surgical
www.quest surgical.com.au

Tel. 1300 786 454 or (07) 5591 2700
E: sales@quest surgical.com.au

www.mercurymed.com