

Antimicrobial Medical Disposable Curtains

Australian Made

Quest Surgical
www.questsurgical.com.au

 **Softmed**

Product Catalogue



E: sales@questsurgical.com.au

PH: 1300 786 454

Call us today

Product Description.

Product Code: SM-MC
Antimicrobial Medical
Disposable Curtains 120gsm

The Softmed Disposable Curtain range has a fresh, clean appearance and provides effective infection prevention and control. Softmed Disposable Curtains are manufactured from high quality, durable and recyclable non-woven polypropylene fabric. Incorporating an eyelet design for easy loading to existing glider hooks, these curtains are quick and simple to change and have a low environmental impact. They are a convenient disposable curtain that provides a better environment for the patient during their hospital stay.

Softmed Disposable Curtains are equipped with an antimicrobial additive, providing staff, patients, and visitors privacy, protection, and other benefits. The antimicrobial protection infused into the product helps prevent the proliferation of product-damaging microbes that may otherwise shorten its lifespan. Stain and odour-causing microbes are also reduced, helping to keep the product fresher and cleaner in-between changing. Importantly, the antimicrobial additive technology does not wash off or wear away, and will continually inhibit potentially harmful microbes that make contact with the surface of the Medical Disposable Curtains. This offers added peace of mind when it is needed most.



Design Principles.

Softmed Antimicrobial Medical Disposable Curtains are made from the following materials:

- *Non-woven polypropylene fabric.*
- *Mesh (for mesh variants).*
- *Eyelets.*

Intended Use.

Softmed Antimicrobial Medical Disposable Curtains are suitable for any setting, including Recovery, ICU, Day Surgeries, Emergency Department, Burn Unit, Dialysis Unit, Medical Centres, and any other healthcare facilities that use privacy curtains.

ANTIMICROBIAL Medical Disposable Curtains.

- *Inhibits and reduces bacterial growth.*
- *Self-auditing labels with unique identification numbers for traceability.*
- *Fire retardant.*
- *Mesh top version available for enhanced airflow.*
- *Reduced laundering and staff safety issues.*
- *Quick changeover, if required between patients in isolation rooms, or get heavily soiled.*
- *100% polypropylene and can be recycled.*

Softmed® Disposable Curtains have been tested to the latest Australian and International Standards.

- *BS 5867-2: 2008 Type B – Flammability requirements specification – Fabrics for curtains, drapes and window blinds.*
- *ISO 20743:2021 Textiles — Determination of antibacterial activity of textile products.*
- *AS 2755.2-1985 Textile fabrics — Burning behaviour, Part 2: Measurement of flame spread properties of vertically oriented specimens.*
- *AATCC 100: 2019 - Test Method for Antibacterial Finishes on Textile Materials.*


Product Range.

The Softmed factory is the first in Australia to produce the full cycle from non-woven fabric raw materials to end-products, with ISO 13485:2016 Certification (Medical Device Manufacturing). At Softmed, we can manufacture to your specifications and colours. Reach out to your local consultant and we will come to you.

Standard Antimicrobial Medical Disposable Curtains.



Antimicrobial Medical Disposable Curtains in Special Prints.

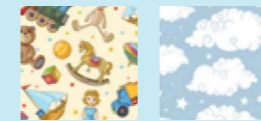

Toy Store Clouds

Antimicrobial Medical Disposable Curtains Mesh Range.

The mesh variant is available in all standard colours and special prints.

Packaging Specifications.

Type	Sizes	Dimensions (m)	Items Per Pack	Packs Per Carton
Normal	Small	2.5 x 2	1	15
	Medium	4.5 x 2	1	8
	Large	7.5 x 2	1	5
With Mesh	Small	2.5 x 2.3	1	15
	Medium	4.5 x 2.3	1	8
	Large	7.5 x 2.3	1	5

Instructions for Use.

Installation of Antimicrobial Medical Disposable Curtains.

1. Perform hand hygiene before hanging medical disposable curtains.
2. Note the date of installation to keep track of the period of use.
3. Keep the disposable curtains in its individual plastic packaging during the installation and ensure that it does not touch the floor.

Installation of Antimicrobial Medical Disposable Curtains.

1. For curtain tracks with existing hooks, simply hook the curtain through the individual eyelets.
2. Softmed Curtain Roller Hooks are available for purchase for curtain tracks employing a runner system.
3. For curtain rails with fixed hook provisions, the Softmed Curtain S-Type Hook can be used for adaptable installation.

Removal and Disposal of Antimicrobial Medical Disposable Curtains.

1. Don a Personal Protective Equipment (PPE) before removing disposable curtains to avoid risk of infection.
2. Hold the curtain away from the body while unhooking, and keep them off the floor.
3. Carefully fold and place the disposable curtains into a large plastic bag for disposal.
4. Disposable curtains that are contaminated with blood or body fluids are disposed of as general waste. The ones not contaminated may be suitable for recycling.
5. Perform hand hygiene after disposing the medical curtains.

Changeover Frequency.

Risk Level	Area	Changeover Frequency
<p>Extreme Risk</p> <ul style="list-style-type: none"> - Outbreak of unknown infectious disease. - Gastroenteritis (e.g. Clostridium difficile, norovirus). - Novel communicable disease (e.g. influenza, SARS-CoV2). - Multi-Resistant Organisms (e.g. CPE/CRE, Candida auris). 	<ul style="list-style-type: none"> - Wards managing patients suffering from a novel or highly transmissible communicable disease. 	<ul style="list-style-type: none"> - Change when visibly soiled or torn. - Change immediately upon discharge of the patient and dispose.
<p>High Risk</p> <p>VRE, MRSA, ESBL.</p>	<ul style="list-style-type: none"> - CICU/PICU/GCNC. - High Dependency Unit. - Haematology/Oncology Units. - Burn Units. - Renal Units. - Operating Suites. - Emergency Rooms. - Wards with patients requiring frequent additional contact precautions. 	<ul style="list-style-type: none"> - Change when visibly soiled or torn. - Six-monthly or as directed by Infection Prevention and Control Team.
<p>Medium Risk</p>	<ul style="list-style-type: none"> - General wards. - Outpatient Clinic Areas. - Medical Imaging Areas. - Rehabilitation. - Sleep Units. - Bear Cottage. 	<ul style="list-style-type: none"> - Change when visibly soiled or torn. - Change annually.
<p>Low Risk</p>	<ul style="list-style-type: none"> - Long-term Care. - Nursing Homes. - Medical Centres. - Radiography Rooms. - Office-based Practices. 	<ul style="list-style-type: none"> - Change when visibly soiled or torn. - Change annually.

Softmed Product Description	Softmed Product Code	UOM per Carton
Standard Options:		
SOFTMED DISPOSABLE MEDICAL CURTAINS SMALL 2.5mx2m BLUE	SM-MC101SBLUE/15	15
SOFTMED DISPOSABLE MEDICAL CURTAINS SMALL 2.5mx2m CREAM	SM-MC101SCREA/15	15
SOFTMED DISPOSABLE MEDICAL CURTAINS SMALL 2.5mx2m GREY	SM-MC101SGREY/15	15
SOFTMED DISPOSABLE MEDICAL CURTAINS SMALL 2.5mx2m LATTE	SM-MC101SLATT/15	15
SOFTMED DISPOSABLE MEDICAL CURTAINS SMALL 2.5mx2m LILAC	SM-MC101SLILA/15	15
SOFTMED DISPOSABLE MEDICAL CURTAINS SMALL 2.5mx2m TEAL	SM-MC101STEAL/15	15
SOFTMED DISPOSABLE MEDICAL CURTAINS SMALL 2.5mx2m TOYSTORE	SM-MC101STOYS/15	15
SOFTMED DISPOSABLE MEDICAL CURTAINS SMALL 2.5mx2m CLOUDS	SM-MC101SCLOU/15	15
SOFTMED DISPOSABLE MEDICAL CURTAINS MEDIUM 4.5mx2m BLUE	SM-MC101MBLUE/8	8
SOFTMED DISPOSABLE MEDICAL CURTAINS MEDIUM 4.5mx2m CREAM	SM-MC101MCREA/8	8
SOFTMED DISPOSABLE MEDICAL CURTAINS MEDIUM 4.5mx2m GREY	SM-MC101MGREY/8	8
SOFTMED DISPOSABLE MEDICAL CURTAINS MEDIUM 4.5mx2m LATTE	SM-MC101MLATT/8	8
SOFTMED DISPOSABLE MEDICAL CURTAINS MEDIUM 4.5mx2m LILAC	SM-MC101LILA/8	8
SOFTMED DISPOSABLE MEDICAL CURTAINS MEDIUM 4.5mx2m TEAL	SM-MC101TEAL/8	8

Softmed Product Description	Softmed Product Code	UOM per Carton
SOFTMED DISPOSABLE MEDICAL CURTAINS MEDIUM 4.5mx2m TOYSTORE	SM-MC101MTOYS/8	8
SOFTMED DISPOSABLE MEDICAL CURTAINS MEDIUM 4.5mx2m CLOUDS	SM-MC101MCLOU/8	8
SOFTMED DISPOSABLE MEDICAL CURTAINS LARGE 7.5mx2m BLUE	SM-MC101LBLUE/5	5
SOFTMED DISPOSABLE MEDICAL CURTAINS SMALL 7.5mx2m CREAM	SM-MC101LCREA/5	5
SOFTMED DISPOSABLE MEDICAL CURTAINS SMALL 7.5mx2m GREY	SM-MC101LGREY/5	5
SOFTMED DISPOSABLE MEDICAL CURTAINS SMALL 7.5mx2m LATTE	SM-MC101LLATT/5	5
SOFTMED DISPOSABLE MEDICAL CURTAINS SMALL 7.5mx2m LILAC	SM-MC101LLILC/5	5
SOFTMED DISPOSABLE MEDICAL CURTAINS SMALL 7.5mx2m TEAL	SM-MC101LTEAL/5	5
SOFTMED DISPOSABLE MEDICAL CURTAINS LARGE 7.5mx2m TOYSTORE	SM-MC101LTOYS/5	5
SOFTMED DISPOSABLE MEDICAL CURTAINS LARGE 7.5mx2m CLOUDS	SM-MC101LCLOU/5	5
Mesh Options:		
SOFTMED DISPOSABLE MEDICAL CURTAINS SMALL 2.5mx2.3m MESH BLUE	SM-MC102SBLUE/15	15
SOFTMED DISPOSABLE MEDICAL CURTAINS SMALL 2.5mx2.3m MESH CREAM	SM-MC102SCREA/15	15
SOFTMED DISPOSABLE MEDICAL CURTAINS SMALL 2.5mx2.3m MESH GREY	SM-MC102SGREY/15	15

Softmed Product Description	Softmed Product Code	UOM per Carton
SOFTMED DISPOSABLE MEDICAL CURTAINS SM2ALL 2.5mx2.3m MESH LATTE	SM-MC102SLATT/15	15
SOFTMED DISPOSABLE MEDICAL CURTAINS SMALL 2.5mx2.3m MESH LILAC	SM-MC102SLILA/15	15
SOFTMED DISPOSABLE MEDICAL CURTAINS SMALL 2.5mx2.3m MESH TEAL	SM-MC102STEAL/15	15
SOFTMED DISPOSABLE MEDICAL CURTAINS SMALL 2.5mx2.3m MESH TOYSTORE	SM-MC102STOYS/15	15
SOFTMED DISPOSABLE MEDICAL CURTAINS SMALL 2.5mx2.3m MESH CLOUDS	SM-MC102SCLOU/15	15
SOFTMED DISPOSABLE MEDICAL CURTAINS MEDIUM 4.5mx2.3m MESH BLUE	SM-MC102MBLUE/8	8
SOFTMED DISPOSABLE MEDICAL CURTAINS MEDIUM 4.5mx2.3m MESH CREAM	SM-MC102MCREAM/8	8
SOFTMED DISPOSABLE MEDICAL CURTAINS MEDIUM 4.5mx2.3m MESH GREY	SM-MC102MGREY/8	8
SOFTMED DISPOSABLE MEDICAL CURTAINS MEDIUM 4.5mx2.3m MESH LATTE	SM-MC102MLATT/8	8
SOFTMED DISPOSABLE MEDICAL CURTAINS MEDIUM 4.5mx2.3m MESH LILAC	SM-MC102MLILA/8	8
SOFTMED DISPOSABLE MEDICAL CURTAINS MEDIUM 4.5mx2.3m MESH TEAL	SM-MC102MTEAL/8	8

Softmed Product Description	Softmed Product Code	UOM per Carton
SOFTMED DISPOSABLE MEDICAL CURTAINS MEDIUM 4.5mx2.3m MESH TOYSTORE	SM-MC102MTOYS/8	8
SOFTMED DISPOSABLE MEDICAL CURTAINS MEDIUM 4.5mx2.3m MESH CLOUDS	SM-MC102MCLOU/8	8
SOFTMED DISPOSABLE MEDICAL CURTAINS LARGE 7.5mx2.3m MESH BLUE	SM-MC102LBLUE/5	5
SOFTMED DISPOSABLE MEDICAL CURTAINS LARGE 7.5mx2.3m MESH CREAM	SM-MC102LCREA/5	5
SOFTMED DISPOSABLE MEDICAL CURTAINS LARGE 7.5mx2.3m MESH GREY	SM-MC102LGREY/5	5
SOFTMED DISPOSABLE MEDICAL CURTAINS LARGE 7.5mx2.3m MESH LATTE	SM-MC102LLATT/5	5
SOFTMED DISPOSABLE MEDICAL CURTAINS LARGE 7.5mx2.3m MESH LILAC	SM-MC102LLILA/5	5
SOFTMED DISPOSABLE MEDICAL CURTAINS LARGE 7.5mx2.3m MESH TEAL	SM-MC102LTEAL/5	5
SOFTMED DISPOSABLE MEDICAL CURTAINS LARGE 7.5mx2.3m MESH TOYSTORE	SM-MC102LTOYS/5	5
SOFTMED DISPOSABLE MEDICAL CURTAINS LARGE 7.5mx2.3m MESH CLOUDS	SM-MC102LCLOU/5	5

Quest Surgical

www.quest surgical.com.au

P O Box 2541 Southport, QLD 4215
Ph: 1300-786-454 F: 5591 8522
E: sales@quest surgical.com.au
ABN 33 118 090 618 ACN 118 090 618



E: sales@quest surgical.com.au

PH: 1300 786 454

Call us today

ABN: 38641094260
131-133 National Boulevard
Campbellfield, Victoria
3061, Australia